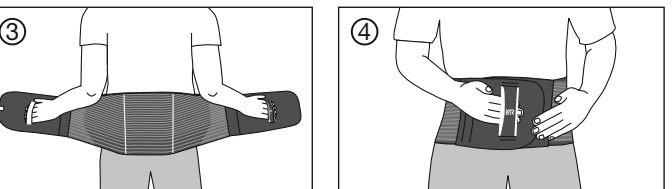
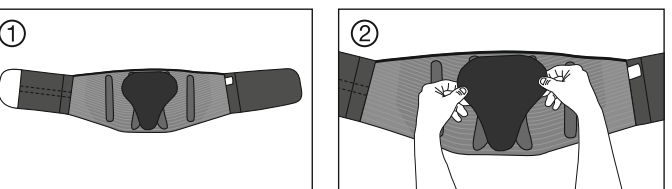
# Cellacare® Dorsal Comfort

Rückenorthese mit Pelotte  
Orthèse dorsale avec pelote  
Back orthosis with insert  
Órtesis dorsal con almohadilla  
Ortesi dorsale con pelotta  
Rugorthese met pelotte

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## en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

### Product description and performance characteristics:

Cellacare® Dorsal Comfort is a back orthosis with a stabilising effect. An integrated x-shaped plate and four stays, together with a removable back insert, contribute to the stability and rigidity of the orthosis. The removable back insert additionally increases stability and pressure in the area of the lumbar region.

### Product composition:

Polyamide, cotton, elastane, polyester, polyurethane, polypropylene, polyethylene

### Intended purpose:

Orthosis for support and stabilisation of the lumbar spine

### Indications:

- Acute or chronic nonspecific lumbar spine pain syndrome caused by incorrect loading (abnormal biomechanical stress) or stress (in terms of load) of muscles and ligaments of the lumbar spine and sacral bone area.
- Acute or chronic specific lumbar spine pain syndrome in the case of
  - Osteochondrosis in the lumbar spine area
  - Spondyloarthritis in the lumbar spine area

### Contraindications:

- Known allergy and / or hypersensitivity to any of the product components
- Severe back injuries / disorders such as instability / fragility of the thoracic and / or lumbar spine with vertebral fractures, in case of bone tumours or metastases or advanced osteoporosis

### Side effects:

In very rare cases, skin irritations and / or allergies may occur.

### Warnings and precautions:

- Do not use Cellacare® Dorsal Comfort unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- The health care professional will determine if the insert is needed, depending on the patient's medical condition.
- Cellacare® Dorsal Comfort shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of pregnancy, the patient's treating physician needs to give the approval before using the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® Dorsal Comfort must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Dorsal Comfort is intended for use on a single patient only.

- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Dorsal Comfort must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

### Use:

#### Preparation:

- Apply Cellacare® Dorsal Comfort while standing.
- For hygienic reasons wearing a vest underneath the orthosis is recommended.
- Open the front hook-and-loop fastener and flatten out the orthosis in front of you (Fig. 1).
- If the removable back insert is separated from the orthosis, attach the back insert to the designated loop area on the inside of the orthosis (Fig. 2).

#### Usage:

- Apply the orthosis around the waist. Make use of the integrated hand grips by putting your right hand through the hand grip marked with the L&R logo and your left hand through the white hand grip (Fig. 3).
- Close the orthosis at the front by using the hook-and-loop fastener. Make use of the integrated hand grip (Fig. 4).
- Check the fit of orthosis and removable back insert. If required, increase or reduce the pressure by pulling harder or less hard when closing the orthosis.

### Washing instructions:

Remove the removable back insert. Take the back insert out of the cover and clean the back insert separately with a damp cloth. Wash the orthosis and the insert cover at 30°C (delicates cycle) with like colours. Do not bleach. Allow Cellacare® Dorsal Comfort to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closure and if possible use a mesh bag. Clean the hook-and-loop closure regularly so it retains its ability to fasten tightly, and to preserve the performance of the product.

### Storage and transport conditions:

Cellacare® Dorsal Comfort must be kept away from light and moisture (preferably in the original packaging).

### Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

### General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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