
# Cellacare® Manu Control Classic

Handgelenkorthese  
Orthèse de poignet  
Wrist orthosis

Órtesis para muñeca  
Ortesi per il polso  
Polsorthese

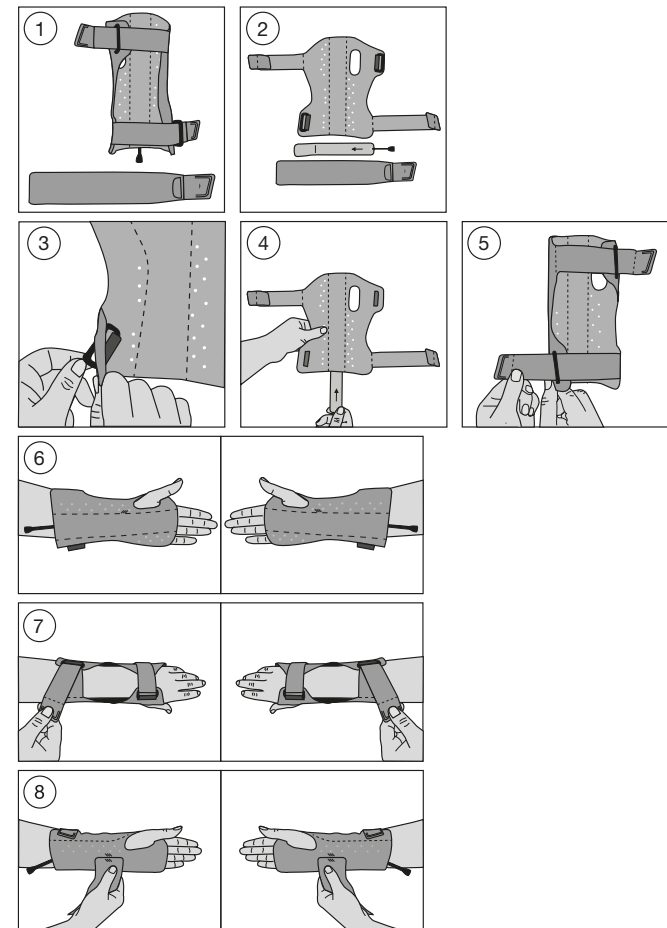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## en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

### Product description and performance characteristics:

Cellacare® Manu Control Classic is an orthosis for the wrist. The combination of two integrated lateral stays, one removable palmar stay and three stabilising straps immobilise the wrist. The stays can be adjusted individually by a specialist if necessary. The orthosis can be worn on the right or left wrist.

### Product composition:

Polyamide, polyurethane, polyester, cotton, polyacetal, aluminium

### Intended purpose:

Orthosis for immobilisation of the wrist

### Indications:

- Posttraumatic treatment of moderate to severe sprains
- Activated osteoarthritis and/or rheumatoid arthritis of the wrist
- Tendinitis in the hand area
- Tenovaginitis
- Treatment of fractures of the wrist after surgery or removal of a cast
- Stage I of complex regional pain syndrome (CRPS)
- Initial carpal tunnel syndrome
- Instability and functional overload in the wrist area

### Contra-indications:

- Known allergy and/or hypersensitivity to any of the product components

### Side effects:

- In very rare cases, skin irritations and/or allergies may occur.

### Warnings and precautions:

- Do not use Cellacare® Manu Control Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Manu Control Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of specific morphological peculiarities in the wrist being treated which restrict or complicate treatment with the orthosis and its correct application, please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Manu Control Classic is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Manu Control Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

## Usage information:

### Usage:

1. Cellacare® Manu Control Classic is configured ex factory for the right wrist.
  - If you are using the product on your right wrist, open the hook-and-loop fasteners on the upper and lower straps and remove the detachable stabilising strap (Fig. 1).
  - Now skip steps 2 to 4 and continue directly to step 1 under the Application section.
  - If you are using the orthosis on your left wrist, please continue with step 2.
2. Open all hook-and-loop fasteners. Pull the upper and lower straps out of the respective buckles and remove the detachable stabilising strap. Remove the stabilising stay from the bottom of the orthosis using the attached cord. There will be minor resistance due to the hook-and-loop fastener at the opening. Flatten out the orthosis in front of you so that the thumb opening is located on the right side of the product (Fig. 2).
3. Push both plastic buckles through the directly adjacent slits so that they are no longer visible from above (Fig. 3). Push the stabilising stay with the L&R symbol on top and in the direction of the printed arrow back into the appropriate opening at the bottom of the orthosis (Fig. 4).
4. Guide the hook-and-loop fastener on the upper strap completely through the upper buckle so that it cannot slip back through. Repeat this process with the hook-and-loop fastener on the lower strap and the lower buckle (Fig. 5).

### Application:

1. Slide the affected wrist into the orthosis so that your thumb goes into the appropriate hole (Fig. 6).
2. Close the strap that goes across your hand by pulling it further through the buckle, and fasten it to the orthosis in the opposite direction using the hook-and-loop fastener. Repeat this process with the strap that goes across the forearm (Fig. 7).
3. Attach the detachable stabilising strap to the underside of the orthosis with the aid of the green positioning mark (Fig. 8). Place the strap over the wrist from the outside and secure it to the underside of the orthosis using the hook-and-loop fastener.
4. Check the fit of the orthosis and the straps and if necessary adjust the pressure on the wrist by adjusting the tension. It is recommended that the palmar stay and the lateral stays be individually adapted by a health care professional to get the best fit.

### Care recommendations:

Remove the palmar stay. Clean the palmar stay separately with a damp cloth. Wash the orthosis at 30 °C (delicates cycle) with like colours. Do not bleach. Allow Cellacare® Manu Control Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closures and if possible use a mesh bag. Clean the hook-and-loop closures regularly so they retain their ability to fasten tightly, and to preserve the performance of the product.

### Storage and transport conditions:

Cellacare® Manu Control Classic must be kept away from light and moisture (preferably in the original packaging).

### Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

### General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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