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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:

Cellacare® Rhizo Classic is an orthosis for the thumb. An integrated stay and two stabilising straps immobilise the articulation carpometacarpalis pollicis (saddle joint of the thumb) as well as articulation metacarpophalangealis I (thumb basal joint), reducing weight bearing and movement. The stay can be adjusted individually by a specialist if necessary. The orthosis can be worn on the right or left thumb.

Product composition:

Polyamide, polyester, polyurethane, cotton, polyacetal, aluminium

Intended purpose:

Orthosis for immobilisation of the thumb

Indications:

- Immobilisation in the case of posttraumatic and postoperative conditions in articulation metacarpophalangealis I (thumb basal joint) and/or articulation carpometacarpalis pollicis (thumb saddle joint)
- Immobilisation in the case of an ulnar ligament injury (skier's thumb)
- Immobilisation in the case of chronic instability of the joints of the thumb
- Immobilisation in the case of degenerative changes in articulation metacarpophalangealis I (thumb basal joint) and/or articulation carpometacarpalis pollicis (thumb saddle joint) (scaphotrapezio-trapezoid arthritis, rhizarthritis)

Contra-indications:

- Known allergy and/or hypersensitivity to any of the product components

Side effects:

In very rare cases, skin irritations and/or allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Rhizo Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Rhizo Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of specific morphological peculiarities in the thumb being treated which restrict or complicate treatment with the orthosis and its correct application, please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.

- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Rhizo Classic is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Rhizo Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information:

Usage:

1. Open the large hook-and-loop fastener at the wider end of the orthosis and the small hook-and-loop fastener at the narrower end of the orthosis. Make sure the wrist strap does not slip out of the buckle. Then slide your hand and thumb into the orthosis (Fig. 1).
2. Pull the large strap around the wrist and affix it to the orthosis by using the hook-and-loop fastener (Fig. 2).
3. Afterwards pull the small strap around the thumb and affix it by using the hook-and-loop fastener (Fig. 3).
4. Check the fit of the orthosis and the straps (Fig. 4) and if necessary adjust the pressure level by adjusting the tension of the straps. It is recommended that the stay be individually adapted by a health care professional to obtain an optimal fit of the orthosis.

Care recommendations:

Wash the orthosis at 30 °C (delicates cycle) with like colours. Do not bleach. Allow Cellacare® Rhizo Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closures and if possible use a mesh bag. Clean the hook-and-loop closures regularly so they retain their ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions:

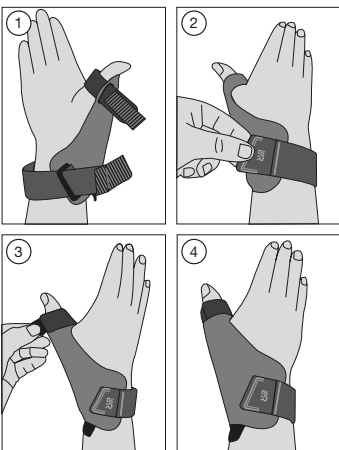
Cellacare® Rhizo Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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