Cellacare® Manus Classic

Handgelenkbandage

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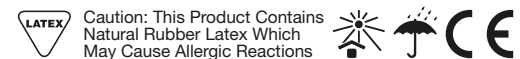
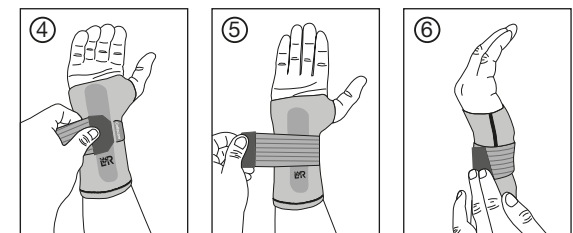
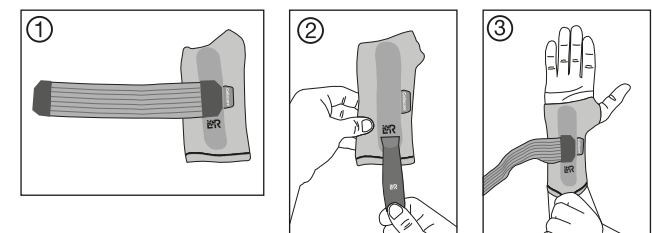
Wrist support

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Polsbandage

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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:

Cellacare® Manus Classic provides compression to the wrist through the combination of the knitted fabric, the crescent-shaped silicone insert and the stabilisation strap and thus reduces the tendency to swell. The wrist joint is stabilised by the anatomically shaped stay (which can be adjusted by a professional if necessary) in a functionally neutral position. If the anatomically shaped stay at the palm is removed, the wrist support's main function will be to provide compression and dynamic stability, which is achieved through the stabilisation strap.

Product composition:

Cotton, elastane, elastodiene (natural rubber latex), polyamide, polyester, polyurethane, silicone

Intended purpose:

Support for compression and stabilisation of the wrist

Indications:

- Posttraumatic treatment of moderate to severe sprains;
- Activated osteoarthritis and/or rheumatoid arthritis of the wrist;
- Tendinitis in the hand area;
- Tendovaginitis;
- Treatment of fractures of the wrist after surgery or removal of a cast;
- Stage I of complex regional pain syndrome (CRPS);
- Instability and function overloading in the wrist area.

Contra-indications:

- Known allergy and / or hypersensitivity to any of the product components

Side effects:

In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Manus Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Manus Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of specific morphological peculiarities in the wrist being treated which restrict or complicate treatment with the orthosis and its correct application, please contact your treating physician / seek medical advice before application of the product.
- In case of circulatory and / or sensory disorders (e.g. with diabetes mellitus); arterial diseases (occlusive arterial disease); venous disease (phlebitis, venous thrombosis), impaired lymph circulation, please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.

- In case of discomfort or problems, consult a health care professional.
- Cellacare® Manus Classic must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Manus Classic is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges.
- Cellacare® Manus Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).
- This product contains natural rubber latex, which may cause allergic reactions.

- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information:

Usage:

1. Before applying the Cellacare® Manus Classic, undo the stabilisation strap (Fig. 1).
2. Grasp the lower edge of the support and pull it over your hand. While doing this, insert your thumb through the thumb opening and your fingers through the large finger opening (Fig. 2). The support must sit closely against the palm.
3. Wrap the stabilisation strap around the wrist and fasten it (Fig. 3 and 4).
4. If you have removed the stabilisation strap from the Cellacare® Manus Classic, fasten the hook-and-loop fastener above the L&R logo, level with the product label (Fig. 5).
5. Removing and re-inserting the removable stay: When re-inserting the stay into the support, look at the marking (L&R logo): this marking must be legible when the product is spread out flat (Fig. 6).

Care recommendations:

Wash the support at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Manus Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Close the hook-and-loop fastener and if possible use a mesh bag. Clean the strap's hook-and-loop closure regularly to ensure that it retains its ability to fasten tightly, and to make sure the product remains effective.

Storage and transport conditions:

Cellacare® Manus Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.