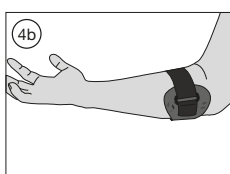
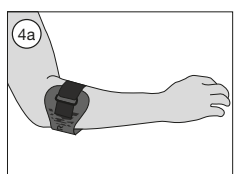
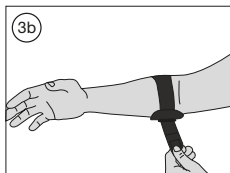
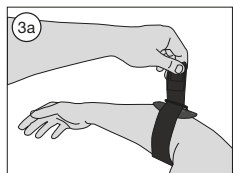
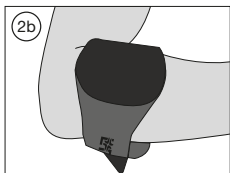
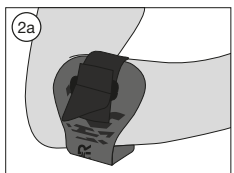
Cellacare® Epi Control Expert

Ellenbogenorthese
Orthèse de coude
Elbow orthosis
Órtesis de codo
Ortesi del gomito
Elleboogorthese

1P einzelner Patient – mehrfach anwendbar · un seul patient – à usage multiple · single patient – multiple use · un solo paciente – uso múltiple · singolo paziente – uso multiplo · één patiënt – meervoudig gebruik

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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics

Cellacare® Epi Control Expert is an orthosis for the elbow. The buckle with integrated silicone insert applies pressure to the muscle tendon origins on the ulnar or radial epicondylus and thus relieves this area. The pressure can be adjusted with a strap.

Product composition

Polyester, polyurethane, viscose, polyamide, polypropylene, cotton, elastane, silicone, polyacetal

Intended purpose

Orthosis to relieve the muscle origins of the ulnar and/or radial elbow (hand extensors and finger extensors or hand flexors and finger flexors)

Indications

- Epicondylitis/epicondylopathia humeri radialis (tennis elbow)
- Epicondylitis/epicondylopathia humeri ulnaris (golfer's elbow)

Contraindications

- Known allergy and / or hypersensitivity to any of the product components

Side effects

- In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions

- Do not use Cellacare® Epi Control Expert unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Epi Control Expert shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of circulatory and / or sensory disorders (e.g. diabetes mellitus); arterial diseases (occlusive arterial disease); venous disease (phlebitis, venous thrombosis), impaired lymph circulation please contact your treating physician / seek medical advice before application of the product.
- In case of specific morphological peculiarities in the elbow being treated which restrict or complicate treatment with the orthosis and its correct application, please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® Epi Control Expert must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Epi Control Expert is intended for use on a single patient only.

- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Epi Control Expert must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information

Preparation

1. Open the hook-and-loop fastener (Fig. 1).

Usage

1. Slide the affected arm into the orthosis. Make sure that the rigid buckle is on the outside of the arm and the strap is on the inside of the arm. The silicone insert must be on the affected area, i.e. :
 - When treating the epicondyle humeri radialis (e.g. for tennis elbow) on the upper side of the forearm (Fig. 2a)
 - When treating the epicondyle humeri ulnaris (e.g. for golfer's elbow) on the underside of the forearm (Fig. 2b)
2. Close the strap by pulling it further through the buckle and affix it to the orthosis in the opposite direction using the hook-and-loop fastener (Fig. 3a/3b).
3. Check the fit of the orthosis (Fig. 4a/4b) and, if necessary, adjust the pressure by loosening or tightening the strap.

Care recommendations

Wash the orthosis at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Epi Control Expert to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop fastener and if possible use a mesh bag. Clean the hook-and-loop fastener regularly so it retains its ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions

Cellacare® Epi Control Expert must be kept away from light and moisture (preferably in the original packaging).

Disposal

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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