Rosidal® soft

Ulcus cruris Kompressionssystem

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en Instructions for Use

Product description and performance characteristics
System for the treatment of venous leg ulcers over a period of approx. 12 weeks.

- For an effective compression bandage, apply 2 Rosidal® soft and 1 Rosidal® K (8 cm and 10 cm) each time. Replacement bandages are included in this set to allow washing after each dressing change. Use the consumables (tg®, Porofil® and Mollelast® haft) as needed (see bandaging instructions, items 1, 9, 16 and 17). If these are used more quickly than expected (e.g. in cases of extensive oedema), the components can also be bought separately.
- The bandage is unrolled with the rolled-up portion (roll) on top. Keep the roll close to the leg. To achieve maximum joint movement, it is recommended to apply the bandage with the foot at a right angle (90°) to the leg.
- Use adhesive strips to fix the bandage in place. Do not fix in place with bandage clips because these can cause injury.

Product composition

- 1x tg® (tubular bandage), size 7, 20 m, 67% cotton, 33% viscose (Ref. 24005)
- 4x Rosidal® soft (foam bandage, white), 10 cm x 2 m x 0.2 cm, 100% polyurethane (Ref. 13016)
- 2x Rosidal® K (short stretch bandage, cream-coloured, 100% CO), 8 cm x 5 m, 100% cotton (Ref. 22201)
- 2x Rosidal® K (short stretch bandage, cream-coloured, 100% CO), 10 cm x 5 m, 100% cotton (Ref. 22202)
- 1x Porofil® (adhesive tape), 2.5 cm x 5 m, Support: 66% viscose, synthetic rubber adhesive (Ref. 143871)
- 1x Mollelast® haft (cohesive fixation bandage, white), 8 cm x 20 m, backing: 66% viscose, 34% polyamide, natural rubber adhesive (Ref. 30070)

Intended purpose

For venous leg ulcer compression therapy

Indications

Venous leg ulcers and their comorbidities, as well as chronic oedema (due to venous insufficiency)

Contraindications

Advanced peripheral arterial occlusive disease, decompensated heart failure, septic phlebitis, phlegmasia coerulea dolens, sensation disorders of the skin, known allergies and/or hypersensitivity to product components.

Side effects

In very rare cases skin irritations and/or allergies may occur.

Warnings and precautions

- Incorrect bandaging can result in skin necrosis and pressure damage to nerves.
- Slight bluish tinge to the toes is normal; this should disappear as soon as the patient starts to walk.
- If the symptoms (pain and bluish tinge) do not disappear, the bandage must be removed and applied again with less compression.
- The patient must inform their doctor if sharply increasing pain occurs while wearing the bandage.
- With an Ankle Brachial Pressure Index (ABPI) greater than 0.8 or with an ABPI of 0.5–0.8 – use only under medical supervision.
- In case of diabetes with microangiopathy advanced, use only under medical supervision.
- The patient must inform the doctor if sharply increasing pain occurs while wearing the bandage.
- Secure the tg® tubular bandage below the knee and the tg® tubular bandage below the heel.
- Return down around the ankle to secure the heel (towards the sole).
- To finish securing the heel, wrap above the ankle (towards the Achilles tendon) and continue to spiral wrap to the base of the calf muscle.
- Wrap the calf using a figure of eight pattern (see illustration).
- Secure the end of the bandage below the heel.
- The finished bandage.

Care recommendations

- Rosidal® K can be washed up to 50 times at 95°C (standard commercially available mild detergent/delicates cycle). Do not dry-safe, do not iron.
- Rosidal® soft can be washed up to 50 times at 95°C (standard commercially available mild detergent/delicates cycle). Do not dry-safe, do not iron.
- Always apply Rosidal® K with full tension (max. stretch).
- It is recommended to apply the bandage in the morning before the patient gets up or after the prolonged period of rest.
- Always apply Rosidal® K with full tension (max. stretch).
- It is recommended to apply the bandage in the morning before the patient gets up or after the prolonged period of rest.

Disposal

In Europe, product waste can be assigned a waste code of Chapter 18 01 of the Directive on the waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance - AV). Recyclable packaging should be recycled according to the relevant national recycling systems.

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