
Velpeau® Ligaction® Pro Expert

Orthèse de genou
Knee orthosis

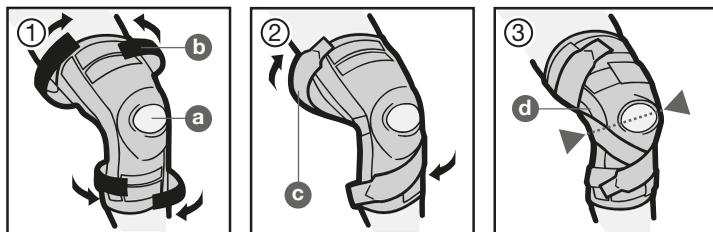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4046481/2022-06

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics

Velpeau® Ligaction® Pro Expert is an orthosis for compression and stabilization of the knee.

Product composition

Cotton, polyamide, polyester, elastane, styrene-butadiene rubber, silicone, polyurethane, aluminum, stainless steel

Intended purpose

Orthosis for compression and stabilisation of the knee

Indications

Treatment of acute and chronic pain and/or irritation of the knee joint, as well as functional deficits after injury or surgery and as a result of incorrect loading or overloading of the knee joint, capsule, tendons, muscles and / or ligaments in:

- moderate knee sprains or contusions
- moderate acute or chronic ligament instability
- moderate patellar hypermobility or instability
- moderate medial or lateral meniscal pathologies (acute traumatic or degenerative)
- postoperative treatment after meniscus surgery

Contra-indications

- Known allergy and/or hypersensitivity to any of the product components

Side effects

In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions

- Do not use Velpeau® Ligaction® Pro Expert unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Velpeau® Ligaction® Pro Expert shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of circulatory and / or sensory disorders (e.g. with diabetes mellitus); arterial diseases (occlusive arterial disease); venous disease (phlebitis, venous thrombosis), impaired lymph circulation in the legs please contact your treating physician / seek medical advice before application of the product.
- In case of pronounced misalignments (genu varum and genu valgum) or individual morphological features of the knee or the legs, which may limit correct application of the orthosis and orthotic treatment please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.

- In case of discomfort or problems, consult a health care professional.
- Velpeau® Ligaction® Pro Expert must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Velpeau® Ligaction® Pro Expert is intended for use on a single patient only.
- The device's durability may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example, when the product is not being worn or when being washed.
- Velpeau® Ligaction® Pro Expert must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information

Place the knee brace on the knee, positioning the kneecap window and kneecap padding over the kneecap (a). Fasten the knee brace using the four semi-circular buckles (b). The two support straps are fixed crosswise.

The round hook and loop closures are attached to the side stays as needed. Different wearing positions are possible, depending on the pathology (c).

Care recommendations

Wash the orthosis at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Velpeau® Ligaction® Pro Expert to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closures and if possible use a mesh bag. The articulated stays can be removed through the horizontal opening by pulling on the side bars. To reinsert the stays, insert the shortest side downwards into the horizontal opening, taking care to ensure that the curved middle section is facing the outside of the orthosis to minimise pressure on the knee joint.

Storage and transport conditions

Velpeau® Ligaction® Pro Expert must be kept away from light and moisture (preferably in the original packaging).

Disposal

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information

In case of a serious incident, contact the manufacturer and the responsible health authorities.

Initial CE mark: 2022