
Cellacare® Gilchrist Easy Classic

Schultergelenkorthese
Orthèse pour l'articulation de l'épaule
Shoulder joint orthosis
Órtesi para la articulación del hombro
Ortesi per l'articolazione della spalla
Orthese voor het schoudergewricht

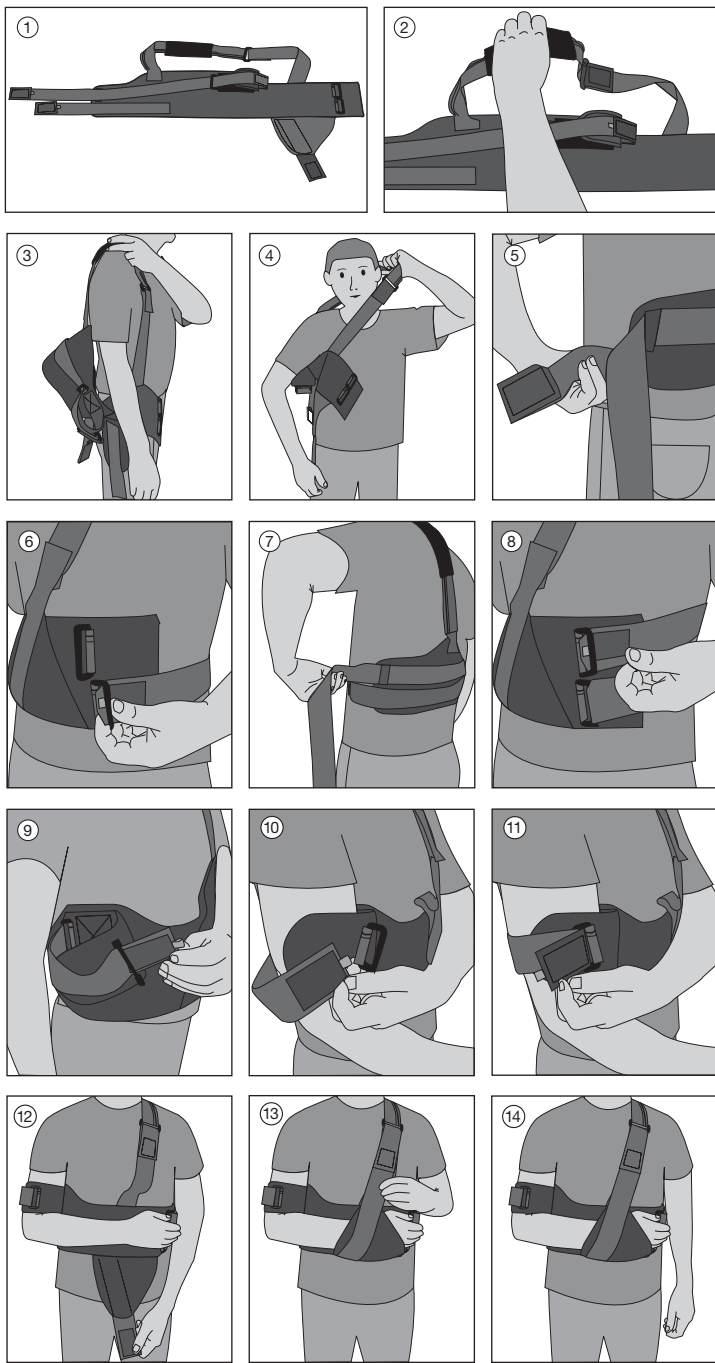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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics

Cellacare® Gilchrist Easy Classic is an orthosis for immobilisation of the joints of the shoulder and the arm.

Product composition

Polyester, Cotton, Polyurethane, Polyamide, Polyacetal (POM), Elastane, Polypropylene

Intended purpose

Orthosis for immobilisation of the joints of the shoulder and the arm in a defined position

Indications

Conservative therapy, post-traumatic and/or pre-/post-operative immobilisation of shoulder and upper arm in the case of:

- shoulder dislocation
- shoulder distortion, shoulder-upper arm contusion
- acute muscle and/or tendon injuries of the shoulder and upper arm
- fracture of the scapula
- fracture of proximal humeral head
- AC joint separation
- paralysis of the upper extremities

Contra-indications

- Known allergy and / or hypersensitivity to any of the product components

Side effects

In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions

- Do not use Cellacare® Gilchrist Easy Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Gilchrist Easy Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of circulatory and / or sensory disorders (e.g. with diabetes mellitus); arterial diseases (occlusive arterial disease); venous disease (phlebitis, venous thrombosis), impaired lymph circulation in the legs please contact your treating physician / seek medical advice before application of the product.
- In case of specific morphological peculiarities in the shoulder being treated which restrict or complicate treatment with the orthosis and its correct application, please contact your treating physician / seek medical advice before application of the product
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Gilchrist Easy Classic is intended for use on a single patient only.
- The device's durability may be compromised by certain factors, e.g., objects with

einzelner Patient – mehrfach anwendbar · un seul patient – à usage multiple · single patient – multiple use · un solo paciente – uso múltiple · singolo paziente – uso multiplo · één patiënt – meervoudig gebruik

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Lohmann & Rauscher International GmbH & Co. KG