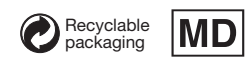
einzelner Patient - mehrfach anwendbar - un seul patient - à usage multiple - single patient - multiple use - un solo paciente - uso múltiple - singolo paziente - uso múltiplo - één patiënt - meervoudig gebruik

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## en Instructions for use

Dear customer,

**Thank you for choosing an L&R product.**

**Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.**

### Product description and performance characteristics:

Cellacare® Lumbal Classic is a back orthosis for stabilisation of the lumbar spine area. An integrated, removable insert and four stays ensure the stability and rigidity of the orthosis. Two elastic additional straps, attached to the front side of the orthosis via the hook-and-loop fastener, complete the stabilising effect and facilitate the adaptable adjustment of the orthosis.

### Product composition:

Polyester, elastane, polyamide, polyurethane, polyethylene, acrylic resin, steel, polypropylene, polyolefin alloy, cotton, viscose.

### Intended purpose:

Orthosis for support and stabilisation of the lumbar spine

### Indications:

- Acute or chronic nonspecific lumbar spine pain syndrome caused by incorrect loading (abnormal biomechanical stress) or stress (in terms of load) of muscles and ligaments of the lumbar spine and sacral bone area.
- Acute or chronic specific lumbar spine pain syndrome in the case of
  - Osteochondrosis in the lumbar spine area
  - Spondyloarthritis in the lumbar spine area

### Contra-indications:

- Known allergy and / or hypersensitivity to any of the product components
- Severe back injuries / disorders such as instability / fragility of the thoracic and / or lumbar spine with vertebral fractures, in case of bone tumours or metastases or advanced osteoporosis

### Side effects:

In very rare cases, skin irritations and / or allergies may occur.

### Warnings and precautions:

- Do not use Cellacare® Lumbal Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- The health care professional will determine if the insert is needed, depending on the patient's medical condition.
- Cellacare® Lumbal Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of pregnancy, the patient's treating physician needs to give the approval before using the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® Lumbal Classic must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Lumbal Classic is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with

sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.

- Cellacare® Lumbal Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

### Usage information:

#### Preparation

1. Apply Cellacare® Lumbal Classic while standing.
2. For hygienic reasons wearing a vest underneath the orthosis is recommended.
3. Open the front hook-and-loop fastener and flatten out the orthosis in front of you. If the insert is separated from the orthosis, attach the insert to the designated loop strip on the inside of the orthosis (Fig. 1).
4. Check that the elastic straps are attached without tension to the dedicated hand grip area (Fig. 3).

#### Usage

1. Apply the orthosis around the waist (Fig. 2). Close the orthosis at the front by using the hook-and-loop fastener (Fig. 3).
2. Check the fit of orthosis and removable back insert. If required, increase or reduce the pressure by pulling harder or less hard when closing the orthosis.
3. Pull the elastic additional side straps forward simultaneously using the hand grips (Fig. 4). Attach onto the large hook-and-loop area, so that there is a pleasant feeling of support in the back area.
4. Check the fit of the orthosis. If required, increase or reduce the pressure by pulling harder or less hard when closing the additional side straps (Fig. 5).

#### Care recommendations

Remove the insert. Clean the insert separately with a wet towel. Machine wash the orthosis at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Lumbal Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closures and if possible use a mesh bag. Clean the hook-and-loop closures regularly so that they retain their ability to fasten tightly, and to preserve the performance of the product.

#### Storage and transport conditions

Cellacare® Lumbal Classic must be kept away from light and moisture (preferably in the original packaging).

#### Disposal

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

#### General information

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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