Cellacast® Soft

Semirigid Stützverband
Bandage de soutien semi rigide
Semi-rigid casting tape

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Contraindicaciones
Known allergy and/or hypersensitivity to any of the product components.

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Side effects
In very rare cases skin irritations and/or allergies may occur.

Warnings and precautions
• Wear gloves during application.
• Only use the product in well-ventilated areas.
• The uncured cast can irritate skin and eyes and sensitisation by inhalation is possible.
• Avoid direct contact with the uncured cast. Any points of contact can be cleaned immediately with alcohol or acetone.
• Advise the patient that bathing with the cast is not recommended because wet padding material under casts can lead to skin maceration. Wet casts can be dried with a towel and a hair dryer.
• Advise the patient that he/she must report any feelings of numbness or pressure immediately and must ensure that no foreign bodies (e.g. sand) get under the cast because they can lead to skin irritation.
• Do not use if the packaging is damaged or inadvertently opened.

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3. Storage and transport conditions
Store Cellacast® Soft casting tapes at a consistent temperature of no more than 30°C in a dry location.

4. Disposal
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (German Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems. When disposing of casting tape, ensure that has not been used, open the packaging so that the product can cure. After at least 5 minutes, the cured product can be disposed of in household waste.

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