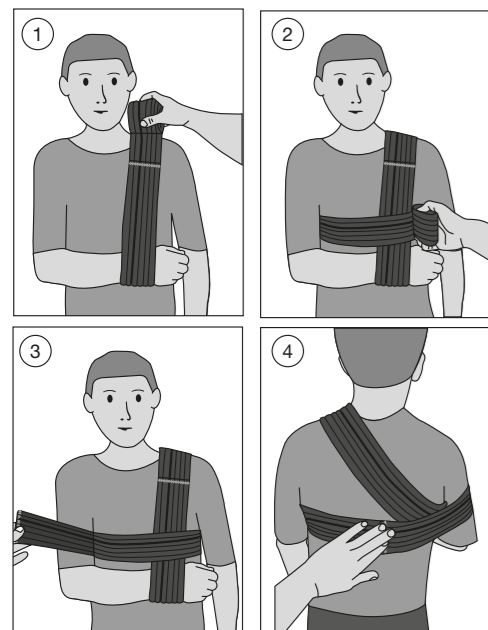
Cellacare® Gilchrist Classic

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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:

Cellacare® Gilchrist Classic is an orthosis for immobilisation of the shoulder joints and the arm. The orthosis can be worn on the right or left arm.

Product composition:

Polyester, Polyamide, Polyurethane.

Intended purpose:

Orthosis for immobilisation of the shoulder joints and the arm in a defined position

Indications:

- painful muscle-tendon disease of the shoulder
- acute slight shoulder instability
- acute treatment of a shoulder luxation
- distortion or contusion of the shoulder
- acute rotator cuff injury
- for postoperative treatment of rotator cuff surgery or shoulder luxation surgery
- acute injury of the AC joint

Contraindications:

- skin diseases and/or injuries in the application area (open, non-intact or injured skin)

Seek medical advice before applying/wearing Cellacare® Gilchrist Classic in the following cases:

- allergic reactions to one or more of the components

Side effects:

Skin irritations and allergies can occur in very rare cases.

Usage information:

1. Apply Cellacare® Gilchrist Classic while standing. The orthosis can be used for the right or left arm. Apply the orthosis with the help of another person.
2. Form a loop and place the hand of the affected arm in the loop (Fig. 1).
3. Pass the strap over the healthy shoulder and pass it diagonally over the back. Pass the strap below the injured shoulder back in front of the body and over the secured hand (Fig. 2).
4. Draw the strap below the healthy shoulder to your back and pass it again below the injured shoulder in front of the body (Fig. 3). Wrap the strap around the upper arm of the injured side and secure it behind the back using the hook-and-loop fastener (Fig. 4).

Warnings and precautions:

- Do not use Cellacare® Gilchrist Classic unless directed by a health care professional. The first time patients use Cellacare® Gilchrist Classic they need to have a health care professional show them how to put it on correctly.
- Apply Cellacare® Gilchrist Classic in accordance with the instructions for use and illustrations provided. Please check the positioning of Cellacare® Gilchrist Classic during use and correct as necessary.

- The health care professional will determine how long Cellacare® Gilchrist Classic is to be worn per day based on the indications and healing progress.
- In case of discomfort or problems consult a health care professional.
- Cellacare® Gilchrist Classic shall only be used on intact skin.
- Wounds have to be covered with a suitable wound dressing before use.
- Cellacare® Gilchrist Classic is intended for use on a single patient only.
- Cellacare® Gilchrist Classic's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fasteners. To prevent this, the hook-and-loop fasteners should always be fastened, for example when Cellacare® Gilchrist Classic is not being worn or when being washed.
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Care recommendations:

Machine wash at 30 °C (delicates cycle) with like colours. Do not bleach. Allow Cellacare® Gilchrist Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean, do not use fabric conditioner. Close the hook-and-loop fasteners and if possible use a mesh laundry bag. Clean the hook-and-loop fasteners regularly so they retain their ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions:

Cellacare® Gilchrist Classic must be kept away from light and moisture (preferably in the original packaging). Cellacare® Gilchrist Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).

Disposal

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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Cellacare® is a registered trademark of Lohmann & Rauscher International GmbH & Co. KG