
Cellacare®

Genucast 0° Classic

Genucast 20° Classic

Knieschiene in 0° Stellung
Attelle de genou en position à 0°
Knee splint at 0 degree of flexion
Férula de rodilla en posición 0°
Stecca per il ginocchio in flessione a 0°
Kniespalk in 0° flexie

Knieschiene 20° gebeugt
Attelle de genou en flexion à 20°
Knee splint at 20 degree of flexion
Férula de rodilla 20° curvada
Stecca per il ginocchio in flessione a 20°
Kniespalk in 20° flexie

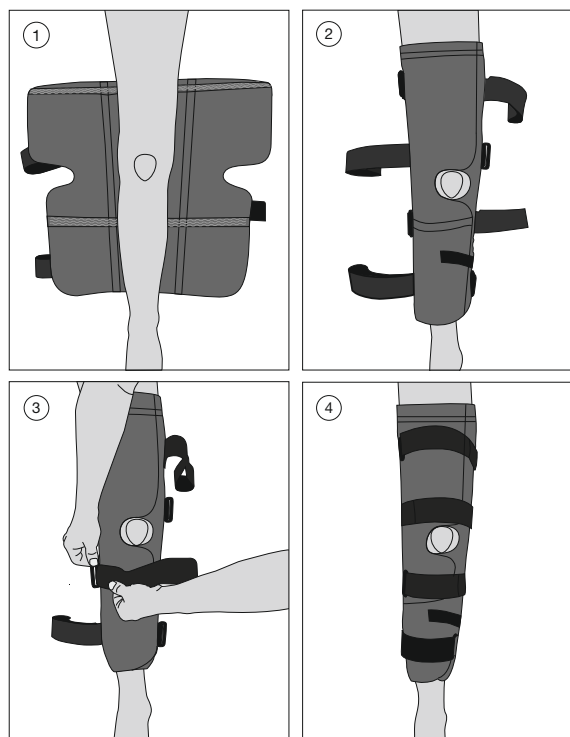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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:

Cellacare® Genucast Classic is a knee orthosis for immobilisation. Conformable aluminium stays help immobilise of the knee joint.

Product composition:

polyester, polyurethane, polyoxymethylene, aluminium, polyamide, silica

Intended purpose:

Orthosis for immobilisation of the knee joint

Indications:

- temporary immobilisation of the knee joint after injuries / operations

Contraindications:

- skin diseases and/or injuries in the application area (open, non-intact or injured skin)

Seek medical advice before applying/wearing Cellacare® Genucast Classic in the following cases:

- allergic reactions to one or more of the components

Side effects:

Skin irritations and allergies can occur in very rare cases.

Usage information:

1. The orthosis should be applied in a lying position.
2. It is recommended that the lateral aluminium stays are individually adjusted by a professional to ensure an optimal fit.
3. Place the orthosis under the leg so that the inner side with silicone bands is in contact with the leg. Make sure that the wider part of the orthosis is placed under the thigh and the narrow part is placed under the lower leg (Fig. 1).
4. Close the orthosis around the leg. Use the provisional fixation band. The front edges of the orthosis overlap. Make sure that the patella is centered in the corresponding opening (Fig. 2).
5. Guide the straps through the opposing buckles (Fig. 3).
6. To ensure the knee splint is positioned correctly, first fix the straps directly above and below the knee. Afterwards fix the other straps (Fig. 4).
7. The straps are adjusted correctly when the orthosis is in tight contact with the leg and when it does not constrict it. If needed, tighten or loosen the straps.

Warnings and precautions:

- Do not use Cellacare® Genucast Classic unless directed by a health care professional. The first time patients use Cellacare® Genucast Classic, they need to have a health care professional show them how to put it on correctly.
- Apply Cellacare® Genucast Classic in accordance with the instructions for use and illustrations provided. Please regularly check the positioning of the product during use and correct as necessary.
- The health care professional will determine how long Cellacare® Genucast Classic is to be worn per day, based on the indications and healing progress.
- In case of discomfort or problems consult a health care professional.
- If local swelling or numbness occurs in the part of the body being treated, Cellacare® Genucast Classic must be removed and a medically qualified health care professional consulted.
- Cellacare® Genucast Classic should only be used on intact skin.
- Wounds must be covered with a suitable wound dressing before using Cellacare® Genucast Classic.
- Cellacare® Genucast Classic is intended for use on a single patient only.
- Cellacare® Genucast Classic's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example, when the product is not being worn or when being washed.
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Care recommendations:

Remove all stays and clean them separately with a damp cloth. Pay attention to how the stays are orientated to ensure they are replaced in the correct way after washing. Wash the orthosis at 30°C (delicates cycle) with like colours. Fasten all hook and loop closures. Do not bleach. Allow Cellacare® Genucast Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Do not use fabric conditioner. Clean the hook-and-loop closure regularly so it retains its ability to fasten securely, and to preserve the performance of Cellacare® Genucast Classic.

Storage and transport conditions:

Cellacare® Genucast Classic must be kept away from light and moisture (preferably in the original packaging). Cellacare® Genucast Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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Cellacare® is a registered trademark of Lohmann & Rauscher International GmbH & Co. KG