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en Instructions for Use

Product Description
Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressings and Packing Rope are soft, conformable wound coverings with a high mannuronic acid content. The calcium alginate fibres react with wound exudate or blood to form a gel which creates a moist wound environment. The high exudate absorption capacity promotes wound healing.

Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressings / Packing Rope are individually sterile packed and sterilised by irradiation. The products are sterile as long as the packaging is unopened and undamaged.

Product composition
Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressings / Packing Rope consist of a homogeneous combination of 20% cellulose fibres and 80% calcium alginate fibres. The alginate fibres are composed of around 60% mannuronic acid and around 40% guluronic acid.

Intended use
Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressings / Packing Rope can be used for the treatment of acute and chronic wounds.

Indications
Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressings / Packing Rope can be used for the treatment of wounds which are:

 heavily exuding

 superficial or

 deep, for

- pressure ulcers
- arterial ulcers
- venous lower leg ulcers
- diabetic ulcers
- post-operative wounds
- skin graft and donor sites

If required, the dressings can be used in conjunction with a suitable moisture-retaining secondary dressing to keep the product in place and/or under compression.

Patient population
Adult patients

Contraindications
Known allergy and/or hypersensitivity to any of the product components. Dry wounds.

Side effects
In very rare cases skin irritations or allergies may occur. Maceration or pain on removing the dressing has been described in isolated cases with this product group.

Application

1. Preparing the wound
Clean the wound carefully in accordance with current clinical standards. The skin surrounding the wound should be clean and dry.

2. Applying the wound dressing

- The size of the dressing depends on the wound size.
- If required, the wound dressing and packing rope may be cut to size with sterile scissors.
- Place the Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressing on the moist wound bed. In the case of slightly exuding wounds, moisten the dressing with physiological saline solution (0,9 %).
- Loosely pack deep wounds with the Suprasorb® A Pro Calcium Alginate Packing Rope.
- Secure the Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressings / Packing Rope with a suitable secondary dressing.

3. Dressing change

The intervals at which the Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressings / Packing Rope must be changed are determined by the treating physician. Depending on the level of exudate, the Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressings / Packing Rope can remain ovn the wound for up to 7 days before being replaced. The dressing is intended to be used for a maximum total treatment period of 30 days.

- Remove the secondary dressing in accordance with current clinical standards.
- Remove the non-gelled part of the Suprasorb® A Pro Calcium Alginate Wound Dressings / Packing Rope and rinse out any gel left behind in the wound with physiological saline solution (0.9%) before assessing the wound and applying a new dressing.

Precautions
Wounds must be assessed and managed appropriately, as indicated, under the supervision of a medical professional. The use of a suitable primary or secondary dressing is at the discretion of the treating physician and depends on the patient's general situation. In the case of wounds or skin lesions, the treating physician will decide on the basis of the patient's general condition whether an occlusive secondary dressing can be applied.

- Application in the case of heavily bleeding wounds, during pregnancy or lactation is decided on by the treating physician. At present, no documented clinical experience with the product in such cases is available.
- Suprasorb® A Pro Reinforced Calcium Alginate Wound Dressings / Packing Rope are for single use only and must not be resterilised.

Storage Instructions
Do not freeze or refrigerate. Store below 25°C.

Disposal
Recyclable packaging should be recycled according to the relevant national recycling systems. Dispose of the dressing in line with current clinical practices for clinical waste.

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Note
Any serious incident that occurs in relation to the Suprasorb® A Pro Reinforced calcium Alginate wound dressings / packing rope should be reported to the manufacturer and competent authority of the relevant established Member State.

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