People.Health.Care.

Rosidal® Lymph – Patienteninformation / Patient Information

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Taking life in stride.

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Congestion in the lymphatic vessels

If there is disruption of the lymphatic system, removal of excess tissue fluid via the lymphatic vessels is impaired. Then swelling occurs in the affected regions of the body. This is known as lymphoedema. Lymphoedema is a chronic disease which can substantially impair the quality of life of those affected.

Various factors can cause lymphoedema. It may develop because of genetic predisposition or after accidents or medical procedures in which parts of the lymphatic system are injured. The extremities, such as the arms and legs, are often affected. To prevent the condition and the symptoms from deteriorating, all available means should be used to treat the disease systematically.

As a patient, you can actively contribute to the success of treatment.

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Complex decongestive therapy (CDT) is divided into two phases: the **decongestive phase** and the **maintenance phase**.

There are four components of the therapy:

- manual lymphatic drainage
- skin care
- exercise
- compression therapy

The last component is considered the most important.

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Manual lymphatic drainage

Manual lymphatic drainage is a special manual technique which is performed by specially trained physiotherapists. It stimulates the pumping action of the lymphatic vessels so that the congested lymphatic fluid is removed.

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Skin care

The skin on the affected part of the body is often very dry and irritated. A moisturising cream or lotion that does not irritate the skin should always be applied to keep the skin supple before the compression dressing is put on.

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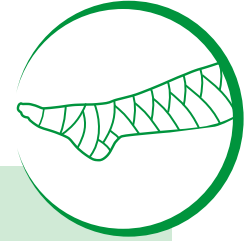
Exercise

Low-key swimming, walking and bicycle rides are ideal ways that you can help with your lymphoedema treatment. It is important not to overdo it. It is better to exercise regularly in moderation. A balanced diet also helps with treatment.

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Compression therapy

Compression therapy is the most important component of lymphoedema treatment. A compression dressing increases external pressure on the tissue to help remove the lymphatic fluid. This reduces the circumference of the affected part of the body. The process consists of two phases.

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PHASE 1: The decongestive phase

The aim of this phase is to reduce the circumference of the affected limb. To achieve this, a compression dressing is applied which helps to remove the congested lymphatic fluid by exerting pressure. Ideally, the compression dressing should be changed daily especially in the initial phase. In this way, the pressure is constantly adjusted to the reducing circumference of the affected part of the body.

L&R offers a solution for every case of oedema. Special emphasis should be put on the Rosidal Lymph Concept which contains all the materials required for professional compression treatment of legs and arms.

An added advantage is that all components of Rosidal Lymph are washable at 95° C and can be reused up to 50 times.

PHASE 2: The maintenance and optimization phase

Phase two aims to maintain the reduction in circumference which has been achieved through the decongestive therapy. To prevent the extremity from swelling again, in this phase of treatment medical compression stockings are used and bring about a lasting improvement in the patient's quality of life.

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With Rosidal Lymph L&R has enhanced its concept of lymphatic compression therapy. Intelligently coordinated components will impress you with their therapeutic efficacy and comfort.